

VETEK



BABY PORTABLE HEIGHT ROD VHM60HM

USER MANUAL



Read this manual carefully before using the instrument

INDEX

1.	INSTALLATION	PAGE 2
2.	INSTRUCTION FOR USE	PAGE 2
3.	INSTALLATION ELECTRONIC HEIGHT ROD WITH BABYONE- BABYTWO	PAGE 4

SAFETY



WARNINGS!

Operators must read this manual carefully, comply with the instructions it contains and become familiar with the correct use and maintenance procedures of the instrument.

- Retain this manual for consultation and as a help in staff training
- Do not apply loads abruptly.
- Do not try to open the instrument.
- Do not remove seals from the instrument.
- Do not short-circuit the battery terminals
- Regularly check the integrity of the instrument's power cord and make sure it does not come in contact with hot appliances
- Do not place the instrument in water or other liquids

By choosing the **WUNDER** mod. **WH80D** professional electronic height rod, you have purchased a high precision instrument.

Since over 40 years **Wunder** has placed its experience at the service of health.


The instrument is characterised by the possibility of fixing the electronic baby scale Wunder model BabyOne or BabyTwo.

The instrument is equipped with a dual LCD electronic terminal with reading to the height of infant.


SPECIFICATION

MEASUREMENT RANGE	35~80cm
GRADUATION	1mm
SIZE	29x7x62cm
WEIGHT	700g


1. INSTALLATION

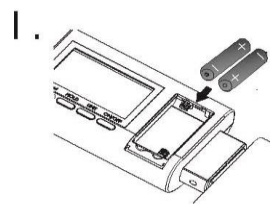


Installing battery

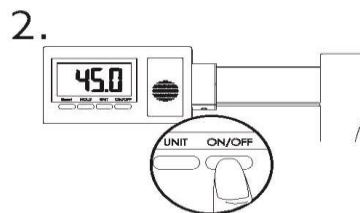


Press ON/OFF to switch on.

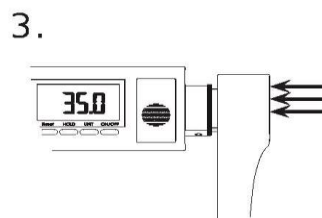

Press right head stop against indicator.

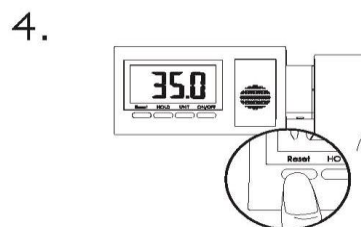

Make sure LCD shows 35.0 cm.

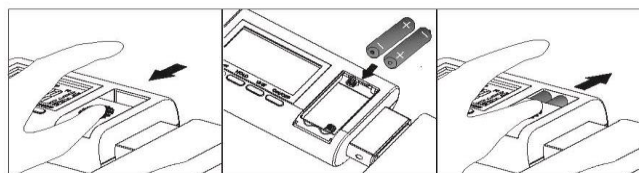


.Batteries installation
1.5V AAA x 2


2. INSTRUCTION FOR USE

KEY FUNCTION	
ON/OFF	To switch On/Off the height rod.
UNIT	To select unit of masuring(cm e inch).
HOLD	To memorize height reading even you slide the rod. It is for user to check the reading again after weighing.Press to memorize, press again to release.
RESET	For user to calibrate false reading. Press right head stop against indicator and press reset.key to calibrate. LCD will show 35,0 cm.

How to measure a baby

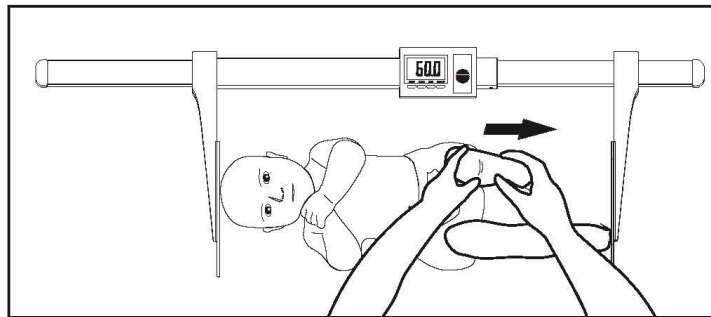

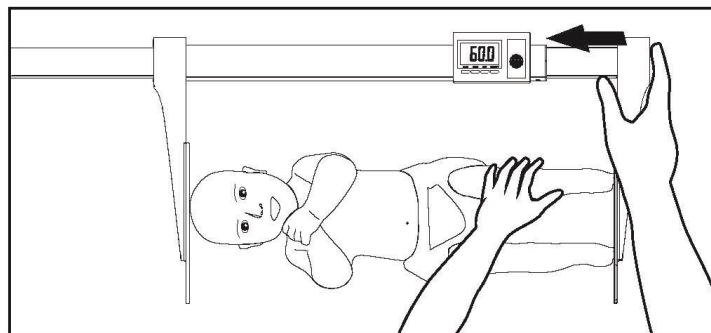

- 1 Lightly press head stop against baby's head
- 2 Straigten baby's feet.



- 3 Sliding right head stop left until it touches baby's feet.
- 4 LCD will display the height of baby.
- 5 Press hold button to memorize the reading of height and write it down.

3. INSTALLATION ELECTRONIC HEIGHT ROD MODEL HM80D TO THE BABY SCALES MOD BABYONE- BABYTWO

For the installation of digital height rod proceed as follow:

<p>A) Apply the support of height rod to the scale's tray through the two screws supplied. (Figure .a).</p>	Fig.A	
<p>B) Insert the digital height rod into the support previously applied to the scale's tray. (Figure.b)</p>	Fig.B	
<p>C) The installation is now complete. (Figurec)</p>	Fig.C	